

CONCENTRATE

XTRA 

STERI-7

**HIGH LEVEL
SURFACE
DISINFECTANT
CLEANER**

**REACTIVE BARRIER
TECHNOLOGY
PROTECTION
BETWEEN CLEANS**

**KILLS PATHOGENS
99.9999%
WITHIN SECONDS**

DILUTION INSTRUCTIONS & HOW TO USE:
 High Level = 1:10 General use = 1:50
 Fogging = 1:10 - 1:50 Wear suitable protective clothing and apparatus where appropriate, refer to MSDS. Average contact time 2 mins, for details of efficacy data please contact info@steri-7.com

HANDLING: Avoid contact with eyes. Do not ingest.

CAUTION: In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. If swallowed, seek medical attention and show this label.

STORAGE: Keep in original containers. Store in a dry place away from direct sunlight between 4°C and 50°

ACTIVE INGREDIENTS: Benzalkonium Chloride <1%, Didecyldimethylammonium Chloride <5, PHMB <1%

APPROVED TO: ISO 13485, ASTM E1052(2000)



For more information please see technical sheet or visit www.steri-7.com or email info@steri-7.com

REF CONS

PCS 95528

STERI-7

34 Baron Grove, Mitcham,
 Surrey, CR4 4EH, England.
www.steri-7.com

CON5.121216.V2.0 GB



USE DISINFECTANTS SAFELY -
Always read the label and product
information before use. Never Mix STERI-7
with other products or chemicals.

- ▶ **REACTIVE BARRIER TECHNOLOGY – PROTECTS BETWEEN CLEANS**
- ▶ **EXCELLENT CLEANING ABILITY**
- ▶ **PROVEN WIDE ANTIMICROBIAL SPECTRUM – BACTERIACIDAL, VIRUCIDAL, SPORICIDAL FUNGICIDAL AND YEASTICIDAL**
- ▶ **RAPID IN ACTION – KILLS UP TO 99.9999% IN SECONDS**
- ▶ **PROVEN ACTIVE IN THE PRESENCE OF ORGANIC MATTER**
- ▶ **DOES NOT DAMAGE MEDICAL DEVICES**
- ▶ **CLEANS AND DISINFECTS AT THE SAME TIME**
- ▶ **FULLY EUROPEAN COMPLIANT**

KEEP OUT OF REACH OF CHILDREN

ANGER



GHS05



GHS09

H315: Causes skin irritation
 H318: Causes serious eye damage
 H411: Toxic to aquatic life with long lasting effects
 P264: Wash hands thoroughly after handling
 P280: Wear protective gloves/protective clothing/eye protection/face protection.
 P302+P352: IF ON SKIN: Wash with plenty of water.
 P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
 P310: Immediately call a POISON CENTER/doctor.
 P321: Specific treatment (see instructions on this label).

LOT

XXXXXX



X.XXXX

